


# ReSound Assist


## Live Assistance Quick Guide

Before your Live Assistance appointment, please ensure:

- You have downloaded the latest version of the ReSound Smart 3D™ app and your hearing aids are paired to your iPhone.
- You are in a location with a strong Wi-Fi connection.
- You insert fresh batteries into your hearing aids. If you have rechargeable hearing aids, make sure they are fully charged.
- You have at least 50% battery on your mobile device.
- Audio routing is set to 'Automatic' to hear audio through the hearing aids or set to 'Never Hearing Devices' to hear audio through the phone speaker. To check your settings open **Settings > Accessibility > Hearing Devices > Audio Routing > Call Audio > select Automatic OR Never Hearing Devices.**

For a full readiness checklist, open the **ReSound Smart 3D app > My ReSound > Live Assistance.**

1. Your hearing care professional will initiate the Live Assistance call. If your phone is locked, swipe to answer the call or tap **Accept**.
2. Tap the Smart 3D icon to enable video access. 
3. Once connected, you will see your hearing care professional on the screen. While your hearing care professional is making adjustments to your hearing aids, you will hear his or her voice through your phone's speaker instead of your hearing aids. At any time, you can select any of the available icons to initiate a chat, turn on or off audio and video or hang up the call.
4. You will see a progress bar appear as your hearing aids are being updated.
5. Once the session has ended, your hearing aids will automatically reboot and you will receive a 'Session ended' notification on your phone. Tap **Close** to enjoy your new settings.

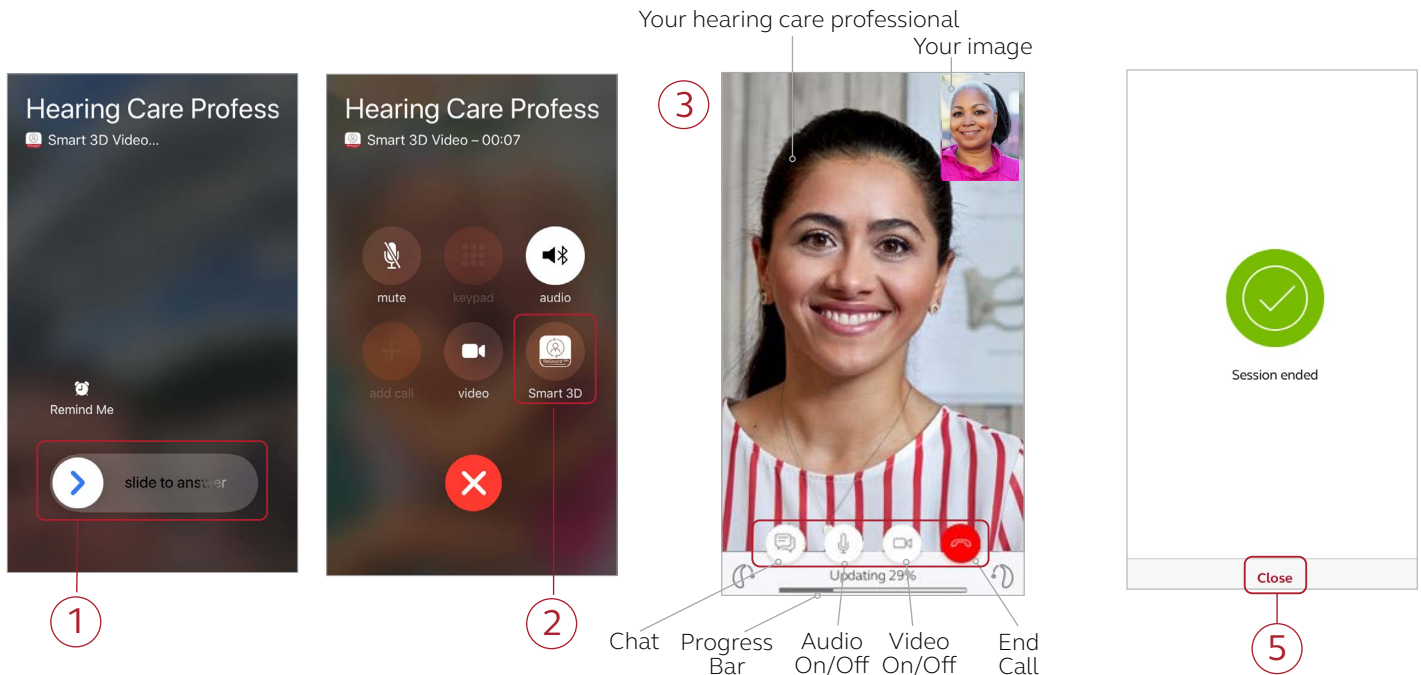

### Manufacturer according to FDA:

#### GN ReSound North America

8001 E Bloomington Freeway  
Bloomington, MN 55420  
USA  
1-888-735-4327  
resound.com

#### ReSound Government Services

8001 E Bloomington Freeway  
Bloomington, MN 55420  
USA  
1-800-392-9932  
resound.com/veterans

### Manufacturer according to Health Canada:

#### ReSound Canada

2 East Beaver Creek Road, Building 3  
Richmond Hill, ON L4B 2N3  
Canada  
1-888-737-6863  
resound.com

