

SureFit 3 RIE Receiver and Dome Overview




Multiple receiver and dome options enable ReSound Receiver-In-The-Ear (RIE) hearing aids to accommodate a wide range of hearing losses.

	MM: Microphone & Receiver-in-Ear (M&RIE)	LP: Low Power	MP: Medium Power	HP: High Power	UP: Ultra Power
Receiver domes					
<p>OPEN (SM, MDM, LGE) Provides the patient with the least occluding experience.</p>	✓	✓	✓	✓	UP receivers require encased molds
<p>TULIP More occluding than open domes. Known to be more comfortable than other domes for some patients. One size option makes it a good choice for demos.</p>	✓	✓	✓	✓	
<p>CLOSED (SM, MDM, LGE) Occlusion similar to Tulip Domes but less occluding than Power Domes. Recommended for use with the M&RIE receivers to help reduce feedback while still providing a solution to occlusion.</p>	✓	✓	✓	✓	
<p>POWER (SM, MDM, LGE) Most occluding dome option for the least risk of feedback.</p>	✓	✓	✓	✓	
GN Wax Filters (Small, White)	✓	✓	✓	✓	


SureFit 3 Receivers

For use with ReSound ONE RIEs

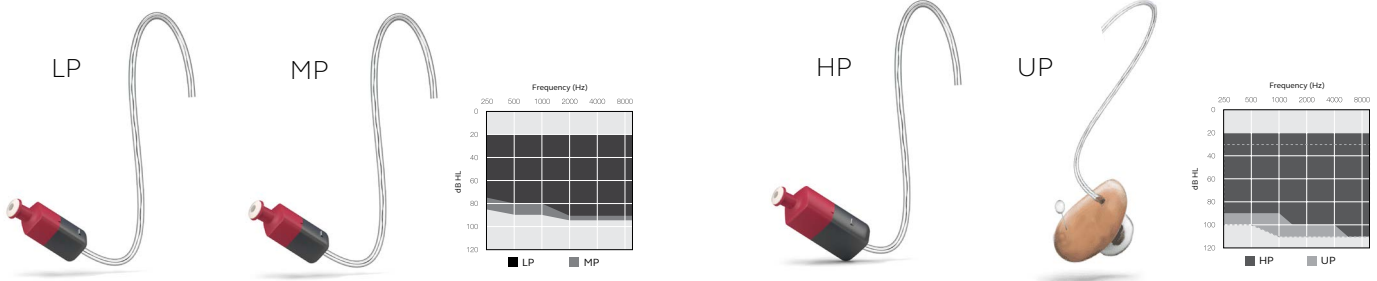

M&RIE Receiver

For use with ReSound ONE RIEs

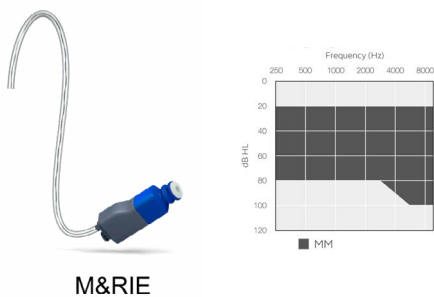

How to replace SureFit 3 and M&RIE receivers



1. Remove the receiver by pushing down with a fingernail on the red or blue receiver lock release found on the belly of the hearing instrument.
2. Receiver lock release will disengage from the belly of the hearing instrument.
3. Pull receiver away from body of hearing instrument.
4. Attach a new receiver by inserting the receiver attachment into the belly of the hearing instrument.
5. Push the receiver attachment until it is flush with the belly of the hearing instrument.
6. The receiver is now attached and ready.

Manufacturer according to FDA:

GN ReSound North America
 8001 E Bloomington Freeway
 Bloomington, MN 55420
 USA
 1-800-248-4327
 pro.resound.com

ReSound Government Services
 8001 E Bloomington Freeway
 Bloomington, MN 55420
 USA
 1-800-392-9932
 gs.resound.com

Manufacturer according to Health Canada:

ReSound Canada
 2 East Beaver Creek Road, Building 3
 Richmond Hill, ON L4B 2N3
 Canada
 1-888-737-6863
 pro.resound.com

