


# Replacing microphone filters

If debris or dust has gathered in the top microphones of the ReSound ONE™ rechargeable hearing aid, you will need to change the top housing. We have an ergonomically designed tool that makes it easy for you to do so, including inserting the new pin.


Only the rechargeable 61 hearing aid model has a field replaceable top housing. You can easily change it by following the instructions below. Microphone cleaning for the 61 (312 battery) and 62 (13 battery) models needs to be carried out at a qualified repair site.

## REPLACING THE TOP MICROPHONE FILTERS ON RESOUND ONE RECHARGEABLE 61 MODEL

### Step 1: Remove the top housing

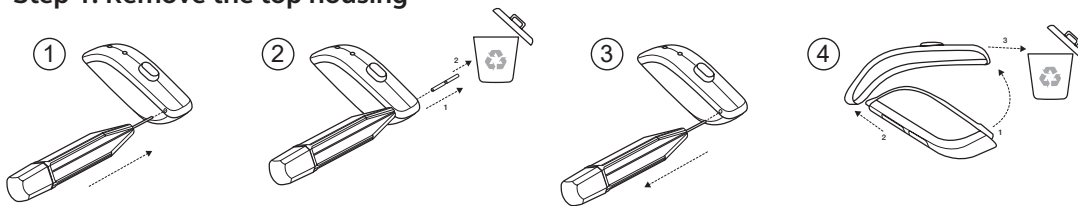


### Step 2: Insert the new top housing

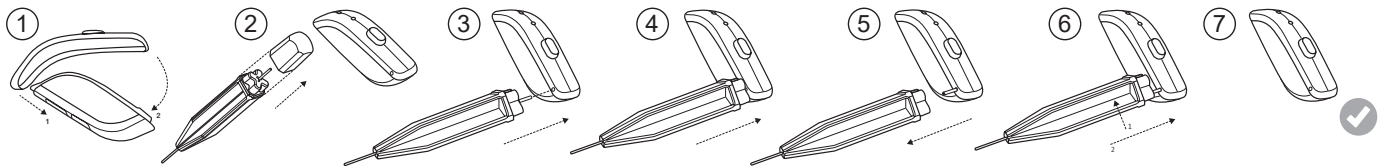

The filter on M&RIE (Microphone & Receiver-In-Ear, labelled MM receiver) can also be replaced in the event of debris in the microphone.

## REPLACING THE MICROPHONE FILTER ON M&RIE

### Step 1: Remove the filter from M&RIE

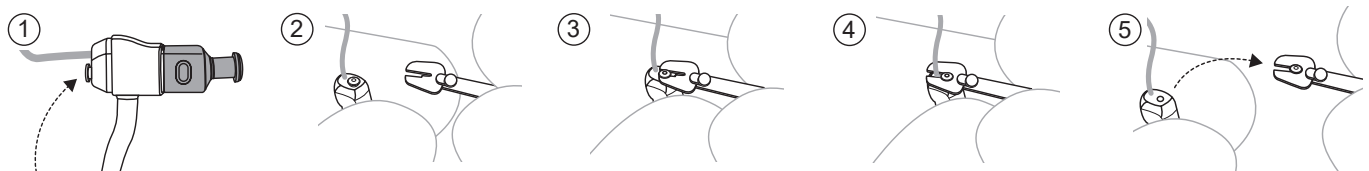

### Step 2: Insert the new microphone filter

