

# 10 easy steps to fit ReSound Key™



A ReSound Key 361 wireless binaural fitting is depicted. Not all features described in this fitting guide are applicable for all ReSound Key fittings.

1. Launch ReSound Smart Fit fitting software and reboot the hearing aids.

2. Click **Connect** in the upper right corner of the Smart Launcher screen.

3. The hearing aids will appear in the ‘Select instruments’ section. Select them by checking the boxes on the left side of the screen.

4. Assign the hearing instruments to the right and left ears by choosing the correct button. It is recommended to beep the instruments to confirm that the instrument selected in the software is in fact the instrument being used in the fitting and on the correct side. Click **Continue** in the lower right of the screen.

5. Select instrument configuration. Click **Continue**. Select the physical properties of the instruments. Click **Continue**. The connection process will now begin.

6. Calibrate the hearing aids to activate DFS Ultra II.



7. Verify connected hearing aids. Click **Continue** to proceed to the Patient Profile screen or pair Accessories by clicking **Accessories**.

8. On the Patient Profile screen, Prescribed First Fit offers customization of a new hearing aid fitting. Based on a user’s audiogram, the selected target rule, gain level (%) and experience level, the ReSound Smart Fit software will adjust the settings for Noise Tracker II, Impulse Noise Reduction and Expansion to better fit the individual user.

- Select the Patient Experience Level based on the amplification history.
- Set Gain Level(%) for initial fit.
- Click **Fit Patient** in the lower right corner to move to Fitting screen.

9. On the Fitting screen, gain adjustments can be made as well as modifications to advanced features. You can also add or remove programs, rename, reorder or recalculate programs under Tools, and access additional tools and features in the upper right menu.

10. Click **Save** to save the fitting to the hearing aids and database.

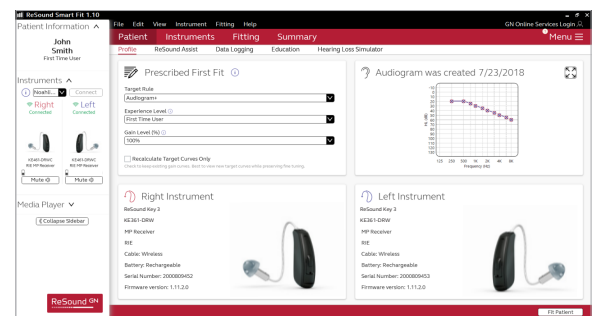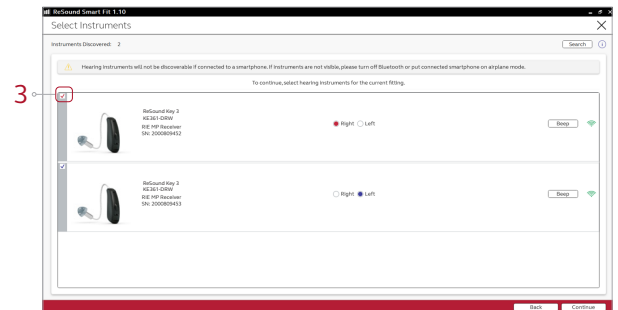

**Manufacturer according to FDA: Manufacturer according to Health Canada:**

**GN ReSound North America**  
 8001 E Bloomington Freeway  
 Bloomington, MN 55420  
 USA  
 1-800-248-4327  
 pro.resound.com

**ReSound Canada**  
 2 East Beaver Creek Road, Building 3  
 Richmond Hill, ON L4B 2N3  
 Canada  
 1-888-737-6863  
 pro.resound.com

