

Charging options for ReSound hearing aids

ReSound offers rechargeable solutions with inductive charging and long battery life to keep your clients on the go.



Premium charger

Standard charger

Desktop charger

Custom charger

ReSound OMNIA™ miniRIE	06	06	06	-
ReSound OMNIA™ BTE ReSound ONE™ BTE	-	-	70	-
ReSound OMNIA™ PBTE ReSound ONE™ PBTE	-	-	80	-
ReSound OMNIA™ ITE/ITC ReSound ONE™ ITE/ITC	-	-	-	C

How to match ReSound hearing aids with the correct charger

Each charger type is offered with different inserts – known as the charging form factor (CFF) – which relate to a specific ReSound hearing aid collection or style.

The CFF is indicated with a number and is shown inside the charger and on the charger packaging.



CFF numbers are not shown on all charger packaging yet, please refer the number inside the charger:

- ReSound One RIE & ReSound Omnia RIE have CFF 1 shown inside the charger
- ReSound Quattro RIE & ReSound Key RIE have CFF 3 shown inside the charger



Charger features

	ReSound OMNIA			ReSound OMNIA ReSound ONE						ReSound LINX Quattro ReSound Key	
	Premium charger	Standard charger	Desktop charger	Premium charger	Standard charger	Desktop charger	Desktop charger	Desktop charger	Custom charger	Premium charger	Desktop charger
	miniRIE	miniRIE	miniRIE	RIE	RIE	RIE	BTE	Power BTE	Customs	RIE	RIE
Inductive charging	•	•	•	•	•	•	•	•	•	•	•
Portable with onboard power bank	•			•						•	
Portable without power bank		•			•				•		
USB-C connector and cable	•	•	•	•	•	•	•	•	•	•	•
LED on the back	Shows the power bank battery status	Lights up when the charger is connected to the main power supply	Shows when the charger is connected to a power supply	Shows the power bank battery status	Lights up when the charger is connected to the main power supply	Shows when the charger is connected to a power supply	Shows when the charger is connected to a power supply	Shows when the charger is connected to a power supply	Shows when the charger is connected to a power supply	Shows the power bank battery status	Shows when the charger is connected to a power supply
LED on the front	Shows the hearing aids charging			Shows the hearing aids charging					Shows the hearing aids charging	Shows the hearing aids charging	

How to tell if the hearing aids are charging

Place the hearing aids in the charger and let the lights indicator on the hearing aids guide you. It is the same process whichever charger is used.

You know that charging has started when the hearing aid lights blink slowly.

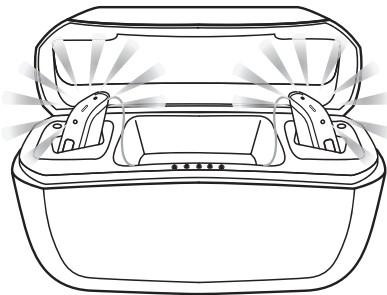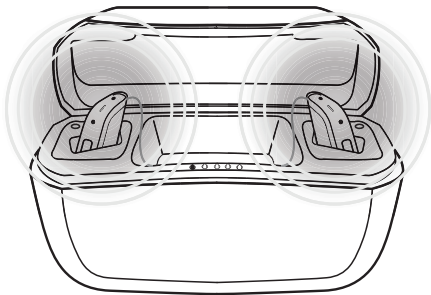

When fully charged, the hearing aid lights remain constant, not blinking.

The lights turn off when the hearing aids are removed from the charger.

Premium charger illustrated.
For more information, please see the individual user guide for each charger.

Worldwide Headquarters
GN ReSound A/S
Lautrupbjerg 7
DK-2750 Ballerup
Denmark
Tel.: +45 4575 1111
resound.com
CVR no. 55082715

United Kingdom
GN Hearing UK Ltd.
Unit 13
Talisman Business Centre
Bicester
Oxon OX26 6HR
Tel: +44 1869 352800
resound.com

Australia
GN Hearing Australia Pty Ltd
Gate C, 19-25 Khartoum Road
Macquarie Technology Park
Macquarie Park NSW 2113
Tel.: (free) 1800 658 955
resound.com

New Zealand
GN Hearing New Zealand Limited
Ground Floor, North Entrance
4 Fred Thomas Drive
Takapuna, Auckland, 0622
Tel.: (free) 0800 900 126
resound.com

Singapore
GN Hearing Pte. Ltd.
456 Alexandra Road
#22-01
Singapore 119962
Tel: +65 6320 9388
resound.com