

How to Change Microphone Filter on rechargeable custom hearing aids

Follow this step-by-step guide to easily replace the microphone filter in clinic on rechargeable custom hearing aids.

To get started, please make sure you have everything needed to replace the microphone filter, including:


1. Service Kit which includes the a tool to remove the microphone filter and the tool to attach a new microphone filter ring to the faceplate.
P/N: 22248700



*Use the side of the tool that is circled



2. New microphone filter rings in the correct color.
P/N: 22799100 LIGHT BEIGE
P/N: 22799102 BROWN
P/N: 22799104 BEIGE
P/N: 22799105 MEDIUM BROWN
P/N: 22799106 DARK BROWN
P/N: 22799111 ANTHRACITE

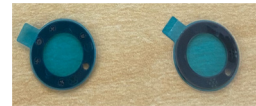

INSTRUCTIONS TO CHANGE MICROPHONE FILTER:



1. Push in the tool in the space between microphone filter ring and faceplate shown in the picture above.



2. Detach microphone filter ring by lifting.




3. Remove the filter ring from the faceplate and discard.



4. New filter rings have a clear sheet on one side and blue sheet on the other. Have the clear sheet facing up.




5. Remove the clear sheet from the filter ring.

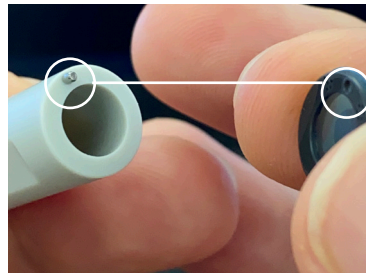


6. Locate the metal pin on the gray cylindrical tool and the hole on the filter ring as shown in the picture above.



7. Place the filter ring on the gray cylindrical tool. Make sure the hole of the filter ring is placed through the metal pin.

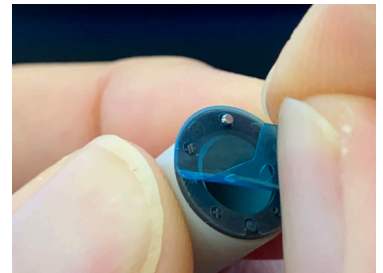

8. Remove the blue sheet from the filter ring.

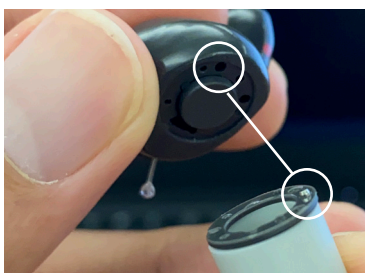

9. Locate the LED on the hearing aids and the metal pin as shown in the picture above.



10. Align LED and metal pin then carefully attach the filter ring to the hearing aid.



11. Press the cylindrical tool firmly to make sure the filter ring is securely attached to the faceplate.



12. This completes the microphone filter replacement.