

## ReSound Remote Control



## Product description

The ReSound Remote Control is for clients who appreciate the ability to easily and discreetly control their hearing aids.

They can use it to change programs and adjust volume, without even looking. This is thanks to a simple keypad that features large buttons, which are concave, tactile and ergonomic. They have raised symbols in the center, so they are easy to locate and glide a finger over. This is especially good for users who struggle with manual dexterity.

The ReSound Remote Control is small and handy and has a streamlined, elegant design. It fits in a handbag, a pocket or the palm of the hand like a keyless car key – and will easily blend in with other smart accessories on a desk or a dinner table.

## **EASY TO CONNECT**

 To pair with a hearing aid, press the pairing button on the back of the ReSound Remote Control and open and close the battery door of the hearing aid or remove the hearing aid from the charger within 20 seconds.

## **EASY TO USE**

- You can change programs in your hearing aid with a simple push on the program button.
- Your hearing aid volume can be adjusted with the + and button.
- The standard CR2032 Lithium coin cell battery is widely available and easy to replace.

Manufacturer according to FDA:	Manufacturer according to Health Canada:
GN ReSound North America 8001 E Bloomington Freeway Bloomington, MN 55420 USA 1-888-735-4327 resound com	GN Hearing A/S Lautrupbjerg 7 DK-2750 Ballerup Denmark CVRno. 55082715
	Distributor in Canada:
ReSound Government Services	ReSound Canada

ReSound Government Services

8001 E Bloomington Freeway Bloomington, MN 55420 USA

1-800-392-9932 resound.com/veterans Resound Carladag 2 East Beaver Creek Road, Building 3 Richmond Hill, ON L4B 2N3 Canada 1-888-737-6863 resound.com

