


Tech Tips

ReSound GN

ReSound Smart Fit | How to Enable Web Updates


As a default, web updates are enabled in ReSound Smart Fit. However, if you previously modified your preferences to turn off web updates, you can re-enable the updates by following these steps:

- Open ReSound Smart Fit and simulate a patient fitting
- Click “Edit” at the top of the screen
- Click “Preferences” in the drop-down menu

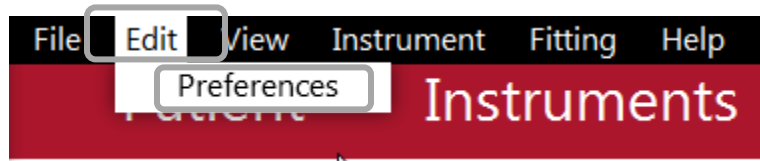

In the Preferences screen, click on “Web Updates”

Check the box next to “Enable Web Updates”.

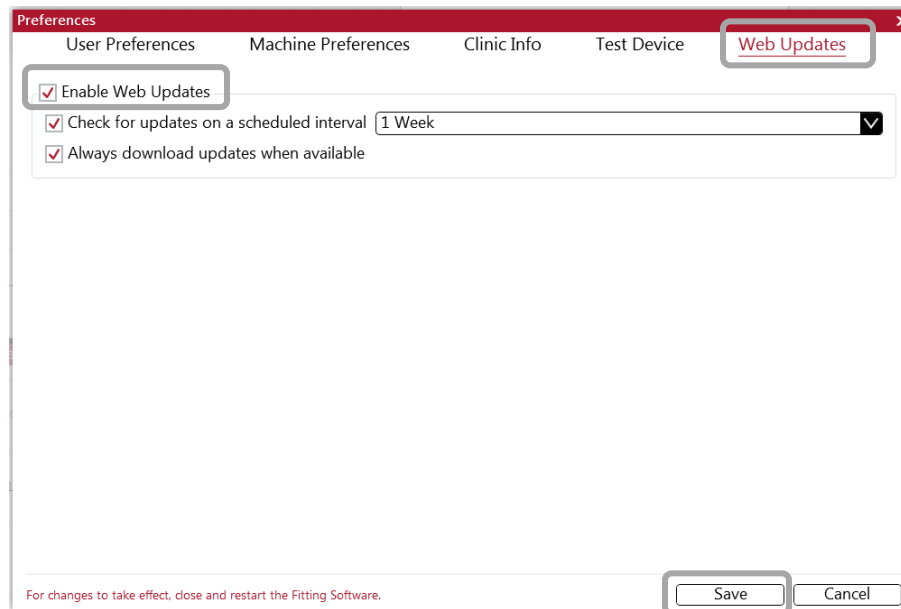

Modify other options as preferred for activating Web Updates and select “Save”.

As instructed in the Web Updates window, close and restart Smart Fit for settings to take effect.

Manufacturer according to FDA:

GN ReSound North America
8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-248-4327
resoundpro.com

Manufacturer according to Health Canada:

ReSound Canada
303 Supertest Road
Toronto, Ontario M3J 2M4
Canada
1-888-737-6863
resoundpro.com