


Made for
iPhone | iPad | iPod

Works with
android

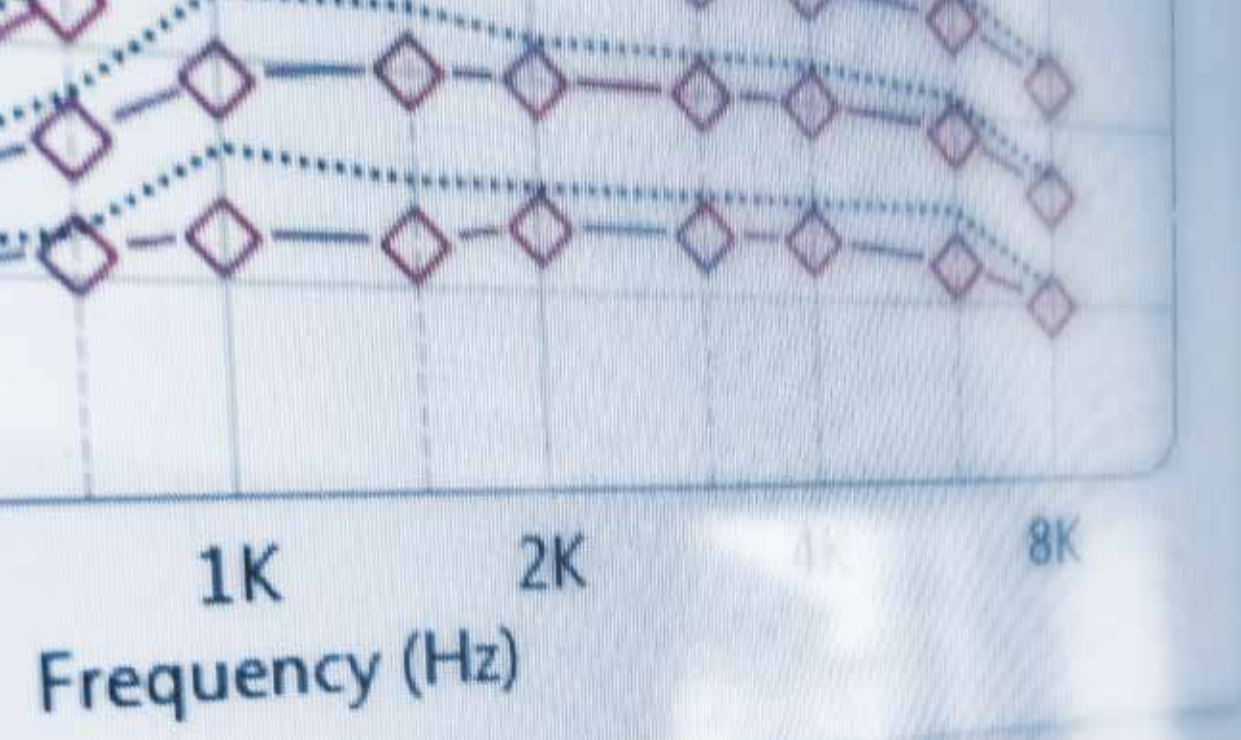

Gain Level
80%

Streaming Accessories



ReSound GN

ReSound LiNX Quattro™

ReSound LiNX Quattro fitting guide

A guide for professionals





First fitting with ReSound Smart Fit™

This fitting guide gives an overview of how to fit ReSound LiNX Quattro wireless hearing instruments with ReSound Smart Fit. A ReSound LiNX Quattro 961 wireless binaural fitting is depicted. Not all features described in this fitting guide are applicable for all ReSound LiNX Quattro fittings.

Prior to connecting to ReSound Smart Fit

ReSound LiNX Quattro Rechargeable 61 RIE

ReSound LiNX Quattro rechargeable hearing instruments can only be connected through wireless interface and cannot be connected with cables. Ensure that the instruments are charged prior to connection. Note that they cannot be connected while in the charger.

Insert Noahlink Wireless in a USB slot on the fitting PC and wait for the message that new hardware has been found. Then open ReSound Smart Fit.

ReSound LiNX Quattro RIEs, BTEs and customs

ReSound LiNX Quattro hearing instruments require fresh batteries when fitting. Remove the stickers from the batteries to activate. Wait for two minutes before using the batteries for best results.

Wireless fittings: Insert a Noahlink Wireless in a USB slot on the fitting computer.


Wired fittings: Ensure that the programming interface (Speedlink, Hi-PRO USB or NOAHlink) is connected.

Launch ReSound Smart Fit. For new fittings, ReSound Smart Fit begins in the Smart Launcher.

Smart Launcher

Both ReSound Aventa and ReSound Smart Fit are contained within one software installation. The Smart Launcher will detect ReSound hearing instruments as well as ReSound Aventa legacy instruments, and then retrieve the software appropriate for the connected device.


1. For ReSound LiNX Quattro rechargeable 961, reboot the devices by placing them in the charger and removing them. Or, reboot the hearing instruments by pressing and holding the push button for 5 seconds until the LED light flashes. This indicates the instrument is 'off'. Press and hold again until the light shows solid, meaning that it is 'on' again.
2. Click 'Connect' in the upper right of the Smart Launcher screen.

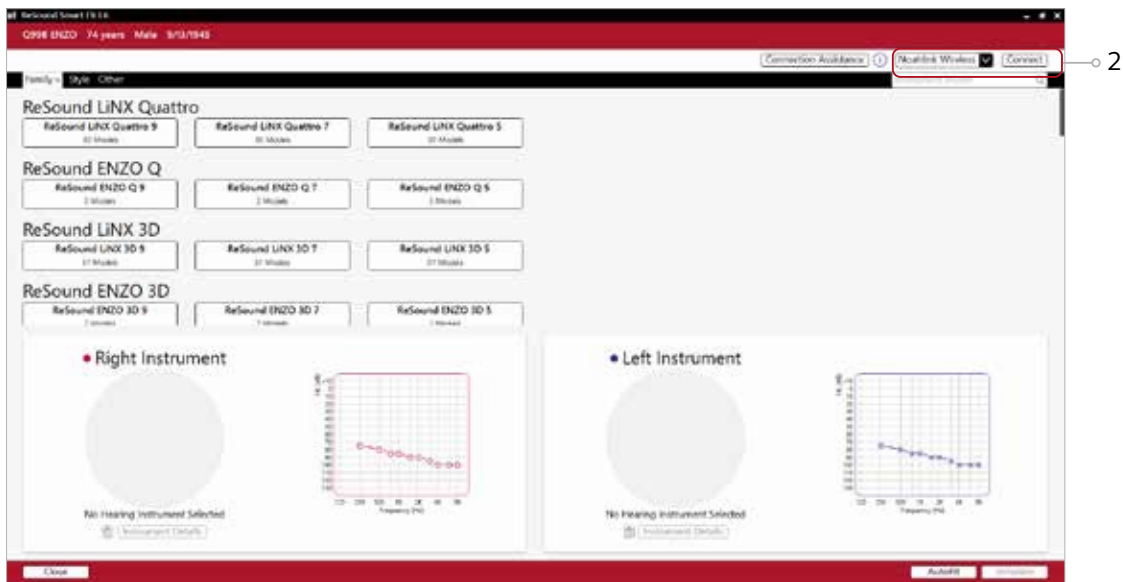

Select instruments

The hearing instruments will appear in the 'Select Instruments' section.


1. When the hearing instruments to be programmed have been identified, select them by checking the left box.
2. Assign the hearing instruments to the right and left ears by choosing the correct radio button for the right and left sides. When pressing 'Beep Instrument', the audio signal will be played and the LED will flash. It is recommended to beep the instrument for confirmation that the instrument selected in the software is in fact the instrument being used in the fitting and on the correct side.
3. Select 'Continue' in the lower right of the screen once the hearing instruments are selected and assigned.

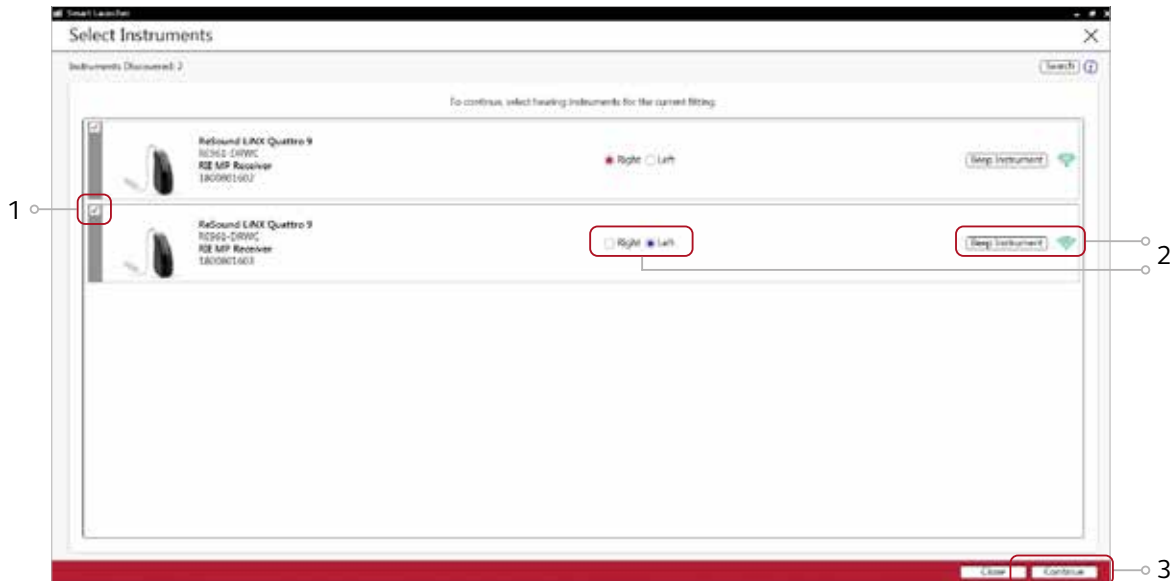


Connection flow

Once the hearing instruments have been selected the connection process will begin.


1. Select instrument configuration and click 'Continue'.

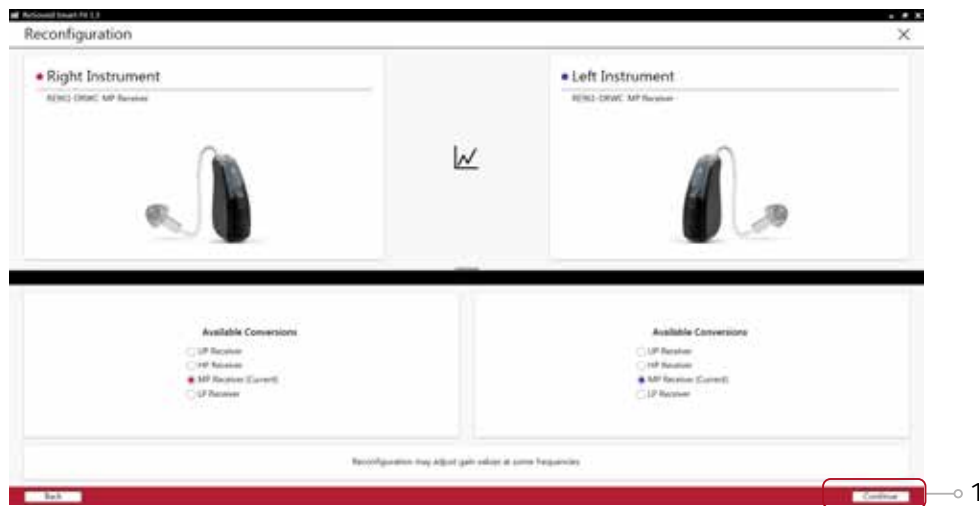


2. Select physical properties of the instrument and click 'Save'.

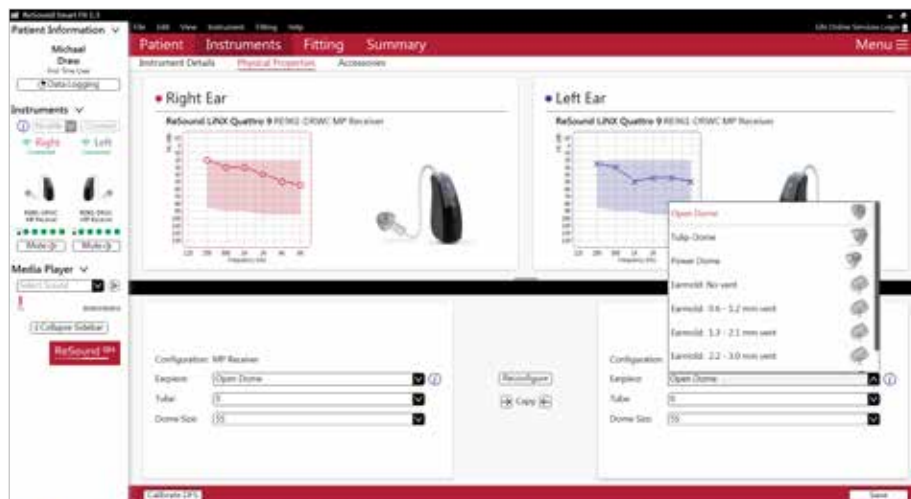

3. Calibration of the hearing instruments is required in order to activate DFS Ultra II. Click 'Calibrate DFS' in the lower left to proceed to the Calibration screen. Inform your patient that he or she will hear a buzzing sound and should sit still and quiet during calibration. Click 'Calibrate' in the lower right corner. When calibration is complete, click 'Continue'.

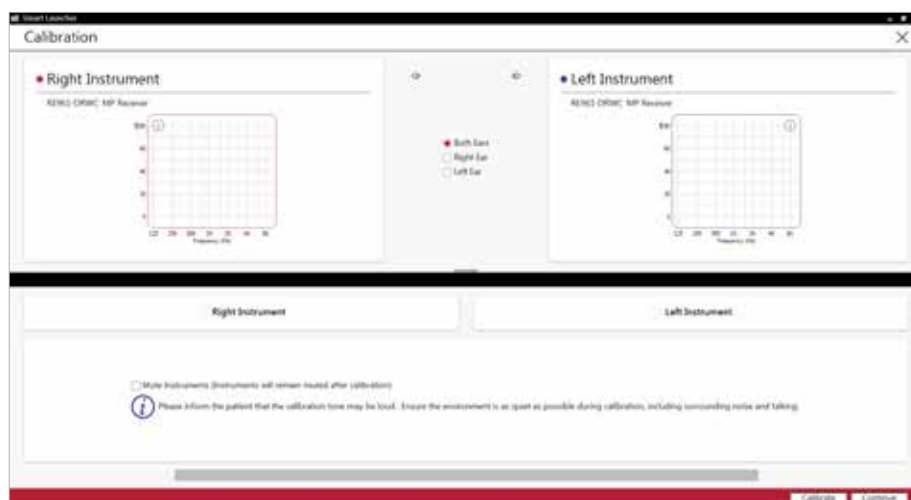


- Verify connected hearing instruments. Click 'Continue' to proceed to the Patient Profile screen or pair Accessories by clicking 'Accessories'.

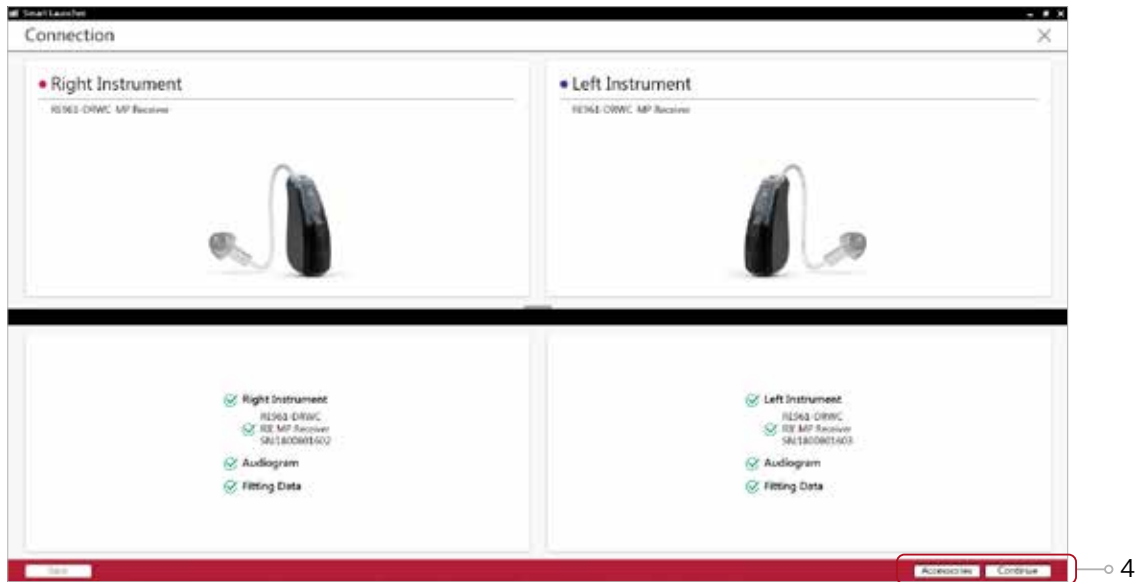


Patient profile screen

- Select the Patient Experience Level based on the amplification history.
- Set the desired programs to be used in the initial fit.
- Click 'Fit Patient' button in the lower right corner to move to Fitting Screen.

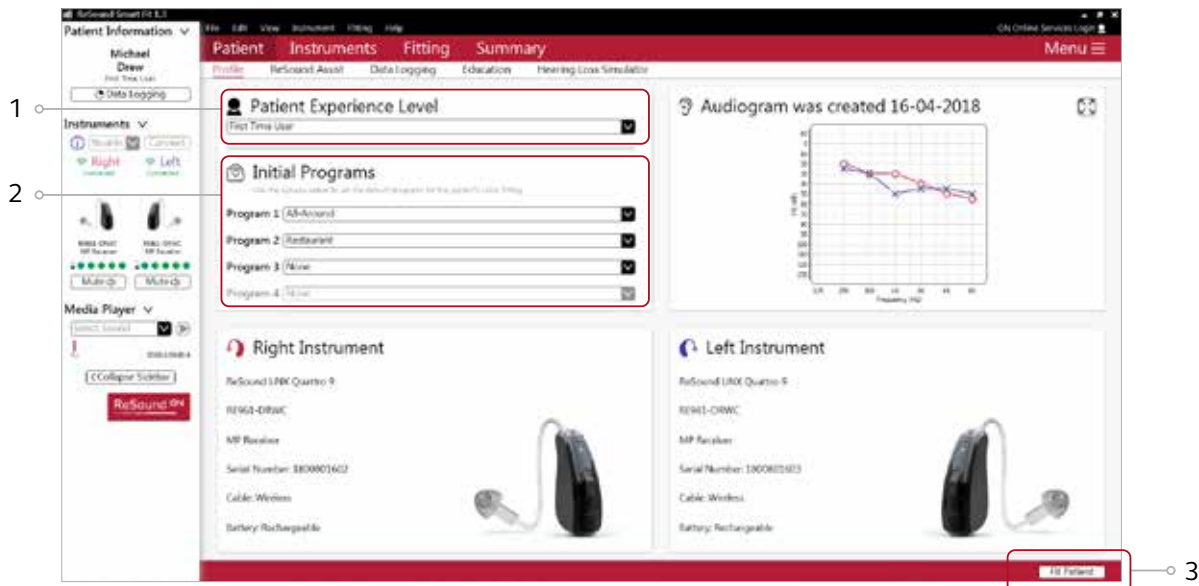

Advanced features

Advanced feature settings can be modified by selecting the appropriate drop down menu or by moving the slider to the desired level. These feature settings apply only to the currently selected program.

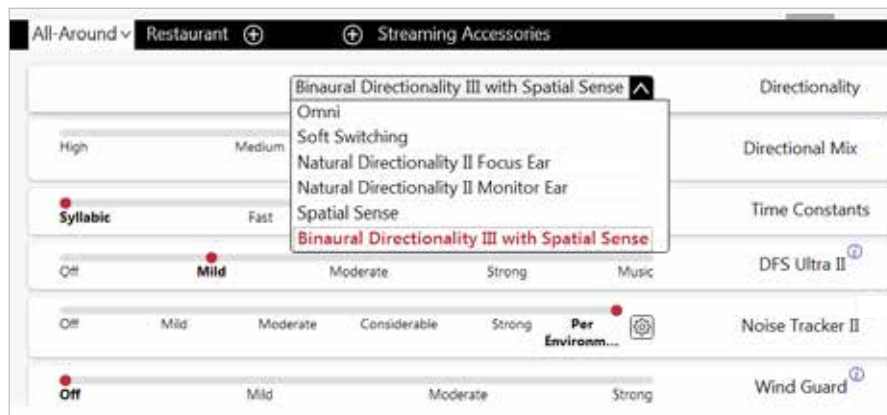

Note: Hover the mouse over each feature for a short explanation of that feature. Clicking a feature with an “i” will redirect to an in depth explanation and video.

Directionality

Binaural Directionality III: Leverages ReSound’s 2.4 GHz wireless technology to allow the hearing instruments to work together, dynamically selecting the best microphone response for the listening environment. Possible configurations are bilateral omnidirectional, bilateral directional or an asymmetric omnidirectional and directional response, depending on the analysis of speech and noise detectors from both hearing instruments. This provides the optimal combination of speech understanding in noise plus a more natural sense of surroundings.

Natural Directionality II: The Focus (directional) ear and Monitor (omnidirectional) ear are prescribed based on the audiogram and any speech audiometry data that may be available.


Spatial Sense: An omnidirectional microphone response which mimics the human ear’s natural response to sound, which preserves spatial cues and helps with the localization of sound sources.

Soft Switching: Seamlessly enables and disables AutoScope/MultiScope Adaptive Directionality™ depending on the listening environment. The response can be either omnidirectional or directional, based on whether noise is present and the direction of speech in the listening situation. With binaurally fit wireless devices at the 9 and 7 technology levels, the hearing instruments synchronize microphone responses.

Auto Scope: Adaptive Directionality enables the beam width to automatically widen or narrow depending on the acoustic surroundings.

Directional Mix

All directional microphone modes have both omnidirectional and directional processing through the frequency spectrum. The setting determines the degree of directionality in the output of the hearing instrument(s). Directional Mix applies omnidirectional processing below a crossover frequency and directional processing above that frequency. The default setting is prescribed depending on the hearing instrument model selected and the degree of low frequency hearing loss. If the patient is still experiencing difficulty hearing in noise increasing the directional mix may help.

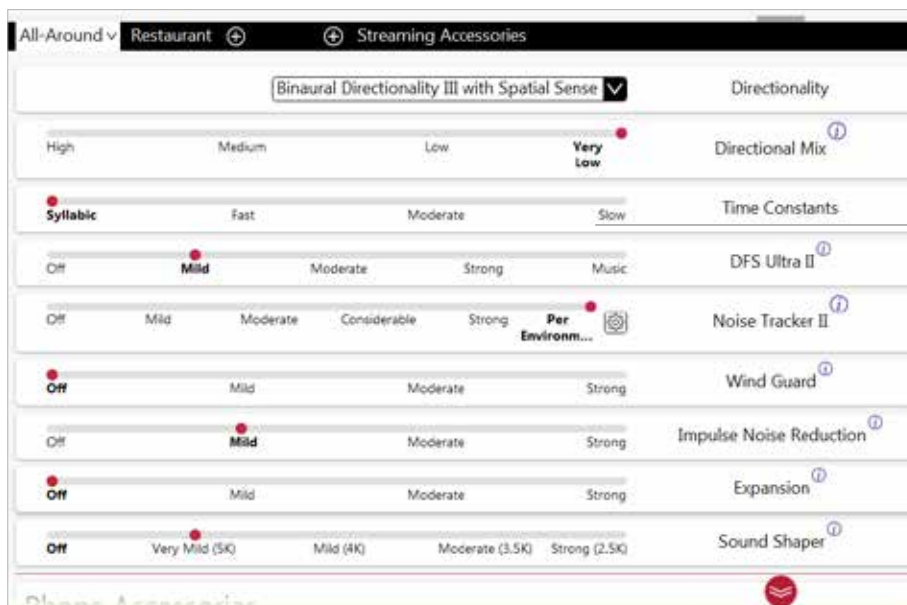


Note: By scrolling down or pulling up the lower part of the Advanced Features screen, all the features available for the connected hearing aid can be viewed.

DFS Ultra II

This is the feedback control. It is activated when the devices have been calibrated for DFS Ultra II. The default setting is Mild. Adjust the setting to a stronger level (Moderate or Strong) if concerns for feedback arise. The Music Mode setting is a Mild setting, typically used for music programs or if the patient experiences feedback only when tonal sounds occur (for example, from a musical instrument).

Noise Tracker II

Noise Tracker II noise reduction uses spectral subtraction to reduce noise between words and syllables in many real-world noisy environments. The Per Environment setting allows for personalized noise reduction settings in seven environments. The environments include Quiet, Soft Speech, Loud Speech, Moderate Speech in Noise, Loud Speech in Noise, Moderate Noise and Loud Noise, and classification is performed on a continuum if the listening situation has characteristics of more than one listening environment. If a patient is reporting specific issues such as hearing conversation in restaurants, for example, the hearing care professional can precisely increase the Noise Tracker II setting only for restaurant-like environments, including Moderate Speech in Noise or Loud Speech in Noise. This maintains the setting for other environments, permitting a great amount of fitting precision and personalization. Alternately, the Mild, Moderate and Strong settings of Noise Tracker II apply the same level of noise reduction for all listening environments, when noise is present.

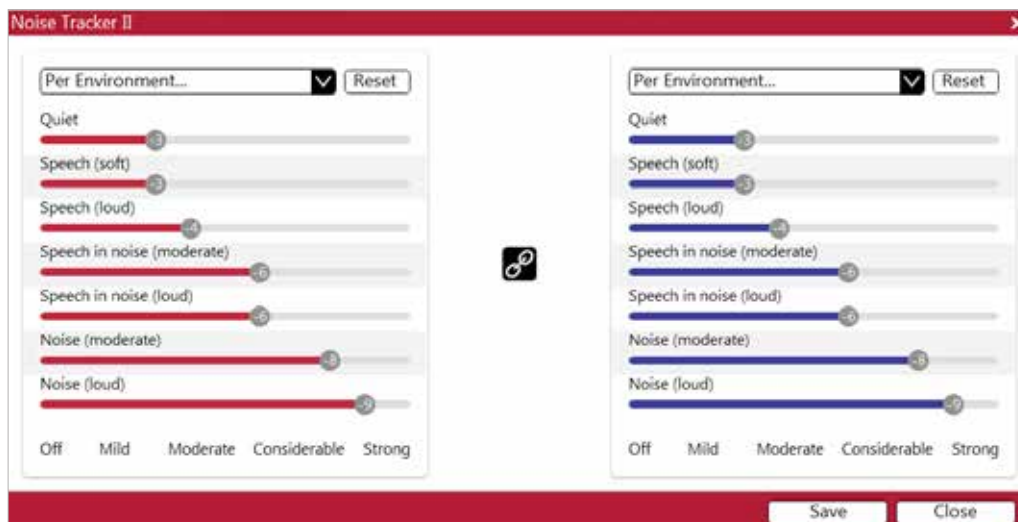

Impulse Noise Reduction

Sudden impulse sounds can be uncomfortable for those using hearing instruments. Impulse Noise Reduction detects the loud impulse sound and attenuates it instantaneously. The feature can be set to three different degrees, with Mild as the default setting. Increasing the setting will decrease the threshold to which the feature will react, or in other words, increase the aggressiveness. Stronger setting will also increase the amount of attenuation.

Wind Guard

Wind Guard applies noise reduction specifically for windy situations. The default setting is off. If a patient is experiencing too much wind noise, increase the setting to a stronger level. If a patient experiences the hearing aid becoming quiet occasionally in outdoor situations, reduce the setting to a milder level.

Auto DFS

Auto DFS is a feedback control setting that is activated automatically before the hearing instrument has been calibrated for DFS Ultra II. Once DFS Ultra II has been calibrated, this setting is grayed out, and cannot be reselected unless the hearing instrument has been restored to factory settings. Auto DFS is designed for use when a patient is unable to return to the clinic for the fitting, and the hearing instrument is instead shipped to the patient from the hearing care professional. It applies a mild level of feedback cancellation, but as it has not been calibrated for the patient's ear, it may not be as effective at controlling feedback as a Mild setting of DFS Ultra II.

Sound Shaper

Sound Shaper applies proportional frequency compression to the fitting. Higher frequencies are lowered to a more usable or audible range for the patient. As patient candidacy criteria for the use of frequency lowering is based on many factors, the default setting is off. However, if the hearing care professional perceives the patient could benefit from frequency lowering in the case of continued difficulties with speech recognition or previous success with frequency lowering features, Sound Shaper can be activated.

The screenshot displays the ReSound Smart Fit 1.3 software interface. The top navigation bar includes 'Patient Information', 'Instruments', 'Fitting', and 'Summary'. The 'Advanced Features' tab is active, showing two frequency response graphs (left and right ears) and a 'Gain Level (%)' slider set to 100%. Below the graphs, a list of settings is visible, including 'Noisy Tracker II', 'Wind Guard', 'Impulse Noise Reduction', 'Expansion', and 'Sound Shaper'. The 'Sound Shaper' setting is currently set to 'Off'. The interface also shows 'Patient Information' for Michael Drew, 'Instruments' (Right and Left), and 'Media Player'.

Low Frequency Boost

Should a patient fitted with an Ultra Power device feel that the sound is not full enough or loud enough, low frequency boost is a quick way to increase the low frequency gain from 250 – 1000Hz.

Phone Accessories

Features related to the streamed sound from a Phone Clip+, or a smart device that is capable of direct streaming, are located at the lower portion of each environmental program tab. Streamed sound will inherit the gain settings of the environmental program that is being used during the time of streaming. While the phone accessory features can be adjusted within the individual program tabs, it is important to note that settings are applied across programs, therefore phone accessory features changes made in one program will also be reflected in others. Please note when viewing Data Logging the time spent streaming from the phone will be collected within the manual program that is being used during streaming.



Environmental Optimizer II

Environmental Optimizer II allows specific gain adjustments for seven different environments, ensuring the user receives optimal gain settings as the acoustic environment changes. Classification is performed on a continuum if the listening situation has characteristics of more than one listening environment. With binaurally fit wireless devices the hearing instruments will optimize and synchronize the Environmental Optimizer II settings for a binaural fitting. If a patient experiences difficulty in a particular listening environment adjust the sliders to assign environment specific gains without requiring a manual program change.




Pairing wireless accessories

The Accessories screen can be accessed in the final step of the connection flow at the bottom of the fitting screen, or in the Accessories tab of the Instruments navigation item. All accessories must be charged or plugged in, and turned on during the pairing process.


1. Click 'Add' beneath the accessory to be paired.



2. Once the searching pop-up appears, press the pairing button on the accessory with a small pointed object such as a pen tip or paper clip. Follow the pairing instructions displayed.

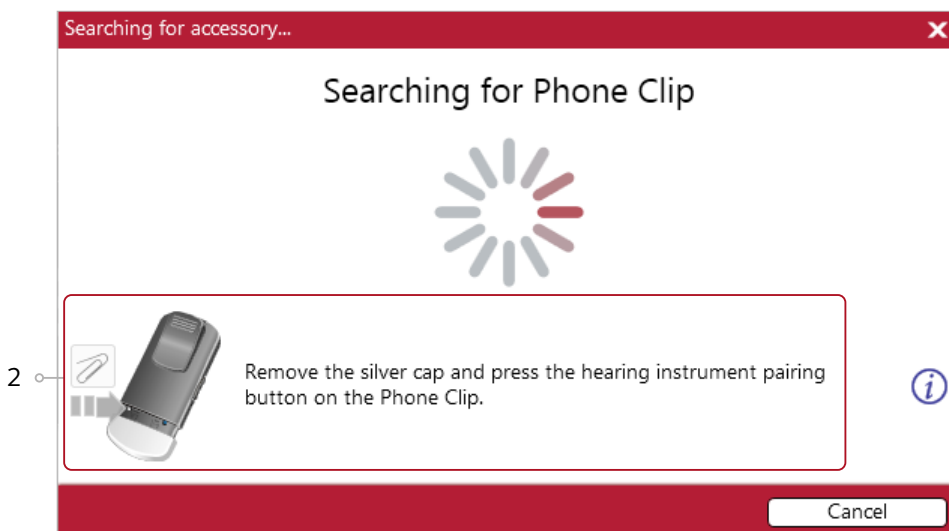

3. When the accessory is successfully paired, the accessory image will appear in color. The 'Add' button will change to a 'Remove' button.

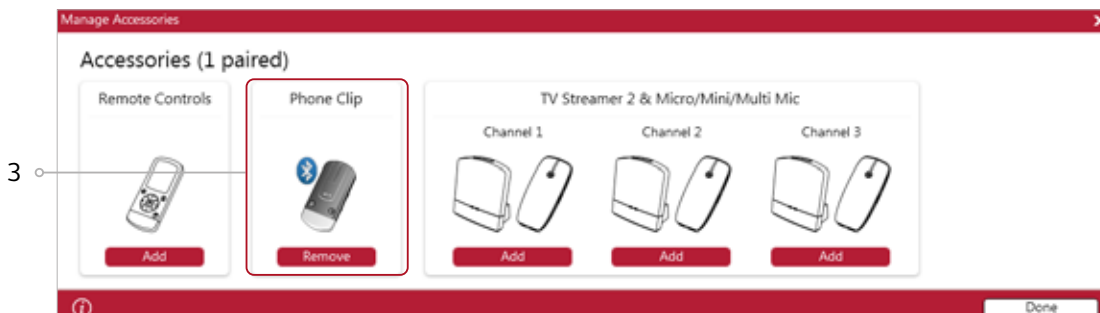

Fitting screen

On the Fitting screen, adjustments can be made to gain, advanced features, and more.

1. Gain adjustments can be made to one or several frequencies by selecting frequencies between 250 Hz and 8,000 Hz on the gain grid and then by clicking the up and down arrows found between the right and left ear grids. Gain can be adjusted for soft, average, and loud input levels which alters the average compression ratios noted above the MPO.
2. Access to advanced features and instrument controls is also found on the Fitting screen. The tabs below the red navigation bar will provide access to these tools.
3. Additional tools and features can be found in the upper right menu.
4. Programs are listed in the program tabs. Select the '+' to add additional programs. Program tools can be found in the tab on the far right.
5. Click the 'Save' button to save the fitting to hearing instruments and in the software.
6. Wireless accessories can be paired both outside and inside the software. To pair inside the software select 'Pair Accessories' to access the accessories screen. Be sure that the accessory is charged or plugged in, and turned on during the pairing process.
7. For LiNX Quattro Rechargeable RIE 61, the power level of the hearing instrument batteries is displayed in the collapsible sidebar. The green dots indicate the amount of charge left in the devices, as also shown on the charger itself.

The screenshot shows the ReSound GN Fitting screen. The interface includes a top navigation bar with tabs for Patient, Instruments, Fitting, and Summary. Below this is a sub-navigation bar with options like Gain Adjustments, Advanced Features, Environmental Optimizer II, and Device Controls. The main area contains two frequency response graphs (one for the right ear, one for the left ear) and a Gain Level (%) slider. A sidebar on the left shows Patient Information (Michael Drew), Instruments (Right and Left), and Media Player. A bottom bar contains buttons for Calibrate OPS, Pair Accessories, and Save. A menu is open on the right, showing options like Patient, In-situ Acoustomy, Speech Scores, Patient Consent, Instruments, AutoFit, Apply Initial Fit, Reconfigure, Test Device, Accessory Updater, Fitting, Tinnitus Sound Generator, Acceptance Manager, AutoREM, Real Ear Measurement, Smart Fit Guide, Feature Demos, Help, Connection Assistance, Other, Sound Player, and Notifications.

Numbered callouts in the image point to the following elements:

- 1: Media Player
- 2: Patient Information sidebar
- 3: Menu button in the top right
- 4: Program tabs (Advanced, Restaurant, Streaming Accessories)
- 5: Save button in the bottom right
- 6: Pair Accessories button in the bottom bar
- 7: Battery status indicator (green dots) in the Instruments sidebar

Completing a follow-up fit with ReSound Smart Fit

Prior to connecting to ReSound Smart Fit

Rechargeable hearing instruments can only be connected through wireless interface and cannot be connected with cables. Ensure that the instruments are charged prior to connection. Note that they cannot be connected while in the charger.

Insert Noahlink Wireless in a USB slot on the fitting PC and wait for the message that new hardware has been found. Then open ReSound Smart Fit. It is not possible for Noahlink Wireless to connect to hearing instruments that are actively connected to a smart device via Bluetooth. Ensure that the hearing instruments do not have an active Bluetooth connection to a smart device. This can be done by having the user turn off their smart device or disable the Bluetooth during the visit. If an active connection to Bluetooth is detected, ReSound Smart Fit will provide a reminder to deactivate the connection with the smart device.

Launch ReSound Smart Fit. For follow up fittings, the software that was used in the previous fitting will launch.




Upgrading a patient from legacy hearing instruments to ReSound LiNX Quattro

ReSound Aventa


If the patient has an existing hearing instrument fit in the latest version of ReSound Aventa, then ReSound Aventa will open when you begin a new fitting.

1. Click the 'Go to Smart Fit' button in the left navigation panel to close ReSound Aventa and arrive at the Smart Fit Launcher.
2. Follow steps in the 'First Fitting' section of this guide

ReSound LiNX Quattro hearing instruments cannot be fit using ReSound Aventa software.



The screenshot shows the ReSound Aventa 3.2.1 software interface. On the left-hand side, there is a navigation panel with several menu items. The 'Go to Smart Fit' button is highlighted with a red rectangular box, and a circled '1' points to it. The main area of the software displays two graphs for 'Right Ear' and 'Left Ear' with various data points and curves. Below the graphs are several tables of data, including a table with columns for frequency (250, 500, 750, 1K, 1.5K, 2K, 3K, 4K, 6K) and rows for different parameters like 'G', 'S', 'C', 'CR', and 'MPO'.

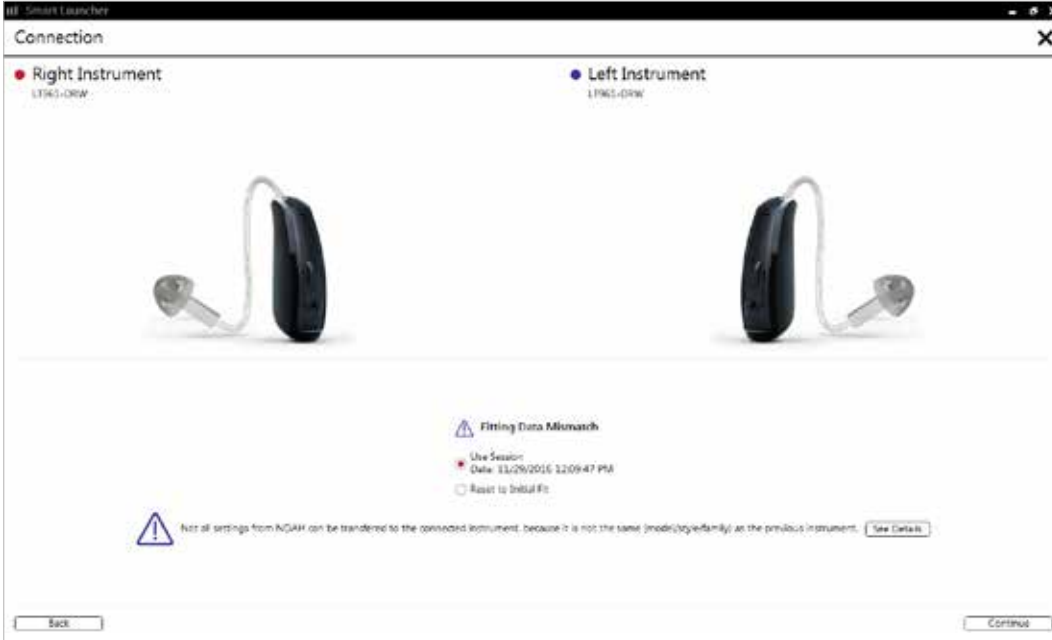

The screenshot shows the 'Smart Launcher' 'Connection' screen. It displays two instrument slots: 'Right Instrument' (L1543-ORW) and 'Left Instrument' (L1965-ORW). Below the instrument slots, there is a warning icon and the text 'Fitting Data Mismatch'. Underneath this warning, there are two options: 'Use Session' (with a date and time) and 'Reset to Initial Fit'. At the bottom of the screen, there are 'Back' and 'Continue' buttons.

Note: To transfer from a previous fitting to a ReSound LiNX Quattro fitting, select 'Use Session' in the fitting data mismatch screen of the connection flow. To start a fresh fitting choose 'Reset to Initial Fit'.

Replacing one hearing instrument in a pair

Connecting during a follow-up

When returning for a follow-up visit the software will open to the last saved session. Depending on the saved preferences, the session will open to either the Patient Profile screen or the Fitting Gain Adjustments screen.

1. For ReSound LiNX Quattro rechargeable 961, reboot the devices by placing them in the charger and removing them. Or, reboot the hearing instruments by pressing and holding the push button for 5 seconds until the LED light flashes. This indicates the instrument is 'off'. Press and hold again until the light shows solid, meaning that it is 'on' again.
2. Click 'Connect' in the collapsible side bar panel.
3. The hearing instruments will appear in the 'Select Instruments' section. When the hearing instrument to be programmed has been identified, select it by checking the left box.
4. The hearing instrument that has already been programmed for this patient will already be assigned to the right or left side. When the other instrument is selected, it will automatically be assigned to the opposite side. Click on 'Beep' to confirm each hearing instrument selected is assigned to the correct side.
5. Click 'Continue' in the lower right of the screen once the instruments are selected and assigned.
6. Continue to connection flow and follow steps in the 'First Fitting' section of this guide.



Note: To transfer settings from a previous session to the replaced instrument, select 'Use Session' in the fitting data mismatch screen of the connection flow. To start a fresh fitting choose 'Reset to Initial Fit'.



At ReSound we understand how important it is for you to have a partner you can trust - one that not only has the latest technology, but also a shared commitment to go that extra mile to help people with hearing loss successfully adapt to life with hearing aids. So it's good to know we've been keeping our promise to help make life sound better for over 150 years, enabling you to help your patients hear more, do more and be more than they ever thought possible.

ReSound is part of the GN Group – pioneering great sound from world-leading ReSound hearing instruments to Jabra office headsets and sports headphones. Founded in 1869, employing over 5,000 people, and listed on NASDAQ OMX Copenhagen, GN makes life sound better.

Find out how you can help your patients get the most out of their hearing experience with ReSound LiNX Quattro at pro.resound.com

 facebook.com/resoundhearing

 twitter.com/resoundus

 youtube.com/resoundus

Manufacturer according to FDA:

GN ReSound North America

8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-248-4327
pro.resound.com

ReSound Government Services

8001 E Bloomington Freeway
Bloomington, MN 55420
USA
1-800-392-9932
gs.resound.com

Manufacturer according to Health Canada:

ReSound Canada

2 East Beaver Creek Road, Building 3
Richmond Hill, ON L4B 2N3
Canada
1-888-737-6863
resoundpro.com

